

# Thermo Wrap Product Line



## IMPACTO® - RSI Protection

*Repetitive strain injuries - RSIs are an everyday concern in the workplace. RSIs can include everything from tendonitis to lower back strain. The IMPACTO Thermo Wrap Line offers a wide range of products for full body RSI Protection.*

## Long Term Wearer Comfort



**TS199**  
Thermo Glove



**TS214 (L), TS215 (R)**  
Wrist / Hand support



**TS216**  
Wrist Support



**TS242 (L), TS243 (R)**  
Carpal Tunnel Brace



**TS205**  
Tennis Elbow Padded



**TS212**  
Elbow Padded




**TS217**  
Elbow support



**TS228**  
Elbow Support w/strap



**TS230**  
Shoulder Support

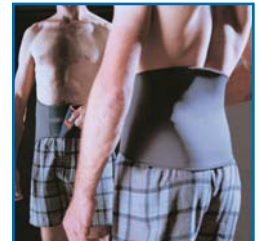

**TS213**  
Back Wrap



**TS209**  
Knee Patella



**TS208**  
Knee Support



**TS866**  
Patella Tracking  
Stabilizer



**TS204**  
Ankle Wrap



**TS226-UREST**  
Rest pad for  
keyboard operators



**IMPACTO®**  
Protective Products Inc.

[www.impacto.ca](http://www.impacto.ca)

E-mail: [impacto@impacto.ca](mailto:impacto@impacto.ca)

Toll free N.America 888-232-0031 Free phone UK 0800 0280 243

# IMPACTO® THERMO WRAPS

**Natural heat therapy aids in the prevention, treatment and rehabilitation of sports injuries, arthritic pain and repetitive strain injuries**

**Multi-layered design with reinforced stitching for durability**

**Lining wicks away moisture and helps prevent skin irritation**

**Non-porous rubberized material captures body heat, increases circulation and provides compression and support**

**Durable nylon outer fabric assures long lasting wear**

**Large range of sizes and styles, washable**

SIZING INFORMATION						
BODY PART	X-SMALL	SMALL	MEDIUM	LARGE	X-LARGE	XX-LARGE
ANKLE Measure around joint of ankle	8¼" - 8½" 21 - 23 cm	8¾" - 9½" 28 - 30 cm	9¾" - 10" 28 - 30 cm	10" - 10½" 28 - 30 cm	10¾" - 11" 28 - 30 cm	11" - 11½" 28 - 30 cm
BACK Measure around middle of waist	23½" - 27¾" 60 - 69 cm	27½" - 31¾" 70 - 80 cm	32" - 35½" 81 - 90 cm	35¾" - 39½" 91 - 100 cm	39¾" - 44" 101 - 112 cm	44¼" - 48½" 113 - 125 cm
ELBOW Measure around elbow joint with arm extended	7½" - 8¾" 18 - 22 cm	9" - 10¼" 23 - 26 cm	10½" - 11¾" 27 - 30 cm	12" - 13¾" 31 - 35 cm	14" - 15¾" 36 - 40 cm	16" - 17¾" 41 - 45 cm
KNEE Measure slightly bent underneath knee cap	10¼" - 11¾" 26 - 30 cm	12" - 13¾" 31 - 35 cm	14" - 15¾" 36 - 40 cm	16" - 17¾" 41 - 45 cm	18" - 19¾" 46 - 50 cm	20" - 21¾" 51 - 55 cm
SHOULDER Measure expanded chest around middle	33" - 35" 84 - 89 cm	Small/Medium 35¼" - 40¼" 90 - 102 cm		Large/X-Large 40½" - 47¼" 103 - 120 cm		47¾" - 51½" 121 - 131 cm
TENNIS ELBOW Measure around elbow joint with arm extended	8¼" - 9" 21 - 23 cm	9¼" - 10½" 24 - 27 cm	11" - 12¼" 28 - 31 cm	12½" - 14¼" 32 - 36 cm	14½" - 16½" 37 - 42 cm	16¾" - 18¾" 43 - 48 cm
WRIST / HAND Measure evenly around wrist joint	4½" - 5¼" 11 - 13 cm	5½" - 6¼" 14 - 16 cm	6½" - 7½" 17 - 19 cm	7¾" - 8¾" 20 - 22 cm	9" - 10" 23 - 25 cm	10¼" - 11¼" 26 - 30 cm

**TS226 is available only in sizes X-Small/Small, Medium and Large/X-Large**

Your THERMO WRAP should fit snug and comfortable. Not too tight, not too loose. If in doubt try the next larger size.

Using the correct size is essential for effective results. Before buying, check size, measurement instructions and application.

To ensure effectiveness, all THERMO WRAPS must be worn directly against the skin.

## Other Thermo Wrap Products Available

Innovative Protection

[www.impacto.ca](http://www.impacto.ca)

ISO 9001:2008

10/13



PO Box 524, Belleville, ON CAN  
 E-mail: [impacto@impacto.ca](mailto:impacto@impacto.ca)  
 Toll free N.America 888-232-0031  
 Free phone UK 0800 0280 243  
 Tel 613-966-0062  
 Fax 613-966-0067

Distributed by,